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POWER SUPPLY
 ATTENTION: RESPECT THE CORRECT CONNECTIO POLARITY.
 NOTE: VERIFY THAT THE VOLTAGE INDICATED ON THE NAME PLATE IS SUITABLE BEFORE ATTEMPTING THE CONNET.

MINIMUM REQUIREMENT
 POWER CORD 3x2,5MM - THE POWER CORD MUST RUN AT LEAST 50CM OUT OF POINT "K".
 THE ELECTRICAL WIRING MUST BE EFFECTIVELY GROUNDED IN COMPLIANCE WITH I.E.C. - US NATIONAL ELECTRICAL CODE - AND C.E.I STANDARDS (OR IN ANY CASE ALL CURRENT LOCAL STANDARDS).
 IN ITALY THE ELECTRICAL WIRING MUST BE DONE IN COMPLIANCE WITH ELECTRICAL CODE C.E.I. 64-8 SEZ.710, THAT REQUIRE A DIFFERENTIAL SWITCH WITH THE FOLLOWING SPECIFICATIONS IS INSTALLED BEFORE THE SYSTEM:

FOR DC X-RAY UNIT WITH POWER SUPPLY 230VAC - 50Hz/60Hz NOMINAL CURRENT: 10A (CURVE D) DIFFERENTIAL SENSITIVITY: 0,03A A CLEARANCE OF AT LEAST 3mm MAXIMUM POWER ANSURPTION: 1,5KWA	FOR AC X-RAY UNIT WITH POWER SUPPLY 230VAC - 50Hz NOMINAL CURRENT: 10A (CURVE D) DIFFERENTIAL SENSITIVITY: 0,03A A CLEARANCE OF AT LEAST 3mm MAXIMUM POWER ANSURPTION: 1,5KWA
---	--

The colour of the 3 wires must be as required by the electrical standard and codes.
 If the differential switch is in a hard-to-reach position place, another switch should be installed that the operator can easult get to.
 The 2 pole power switch should have a clearance of at least 3mm between the contacts, a capacity of 250V 6A and meet the essential requirements of electrical codes I.E.C. 61058-1, 60447 or in any case local codes and standards currently in force.

SIGNAL LAMP CONNECTION
 An independent power suppli should be provided with 2 cables 1,5mm². A conduit system is required to connect the wall plate to a lamp with 2 cables 0,5/1mm².
 The electric connections must be in compliance with all the relevant national regulation.
 WARNING: Look on the preinstallation notes on the technical manual.

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